


Sample requirements

Sample must reach laboratory within 48 hours.

	Preanalytical				Temperature	Required informations	Test request form
	Blood	Bone marrow	Lymph nodes	Other			
Cytology / Cytochemistry	2 slides unfixed and unstained	≥4 slides unfixed and unstained			Room temperature Do not freeze	- Clinical data - FBC	B8-INTGB Malignant blood disorders + BW12-INTGB Fish request form for Malignant Hemopathies and Solid Tumors
Immunophenotyping	1 EDTA tube (or 1 heparin tube) 2 slides unfixed and unstained	2 ml in EDTA (or heparin) 2 slides unfixed and unstained			Room temperature Do not freeze	- Clinical data - FBC	
Conventional cytogenetics : Haematological karyotyping	5ml 1 lithium heparin tube	2-5ml 1 heparin medullary puncture transferred to 1 lithium heparin tube	Vial with sterile culture medium	Biological fluid	Room temperature Do not freeze	- Clinical data - FBC - Bone marrow report - Immuno-phenotyping report	
Molecular cytogenetics : FISH	5ml 1 lithium heparin tube	2-5ml 1 heparin medullary puncture transferred to 1 lithium heparin tube	Vial with sterile culture medium	FISH also available on: unstained slides (peripheral blood or bone marrow smear) paraffin embedded lymph nodes No analysis from bone marrow biopsy	Room temperature Do not freeze		
Molecular biology	Whole blood 2 EDTA tubes	Bone marrow transferred to 1 EDTA tube			Room temperature Do not freeze		
All the preanalytical information can be found on www.eurofins-biomnis.com > Test Guide							



Biomnis

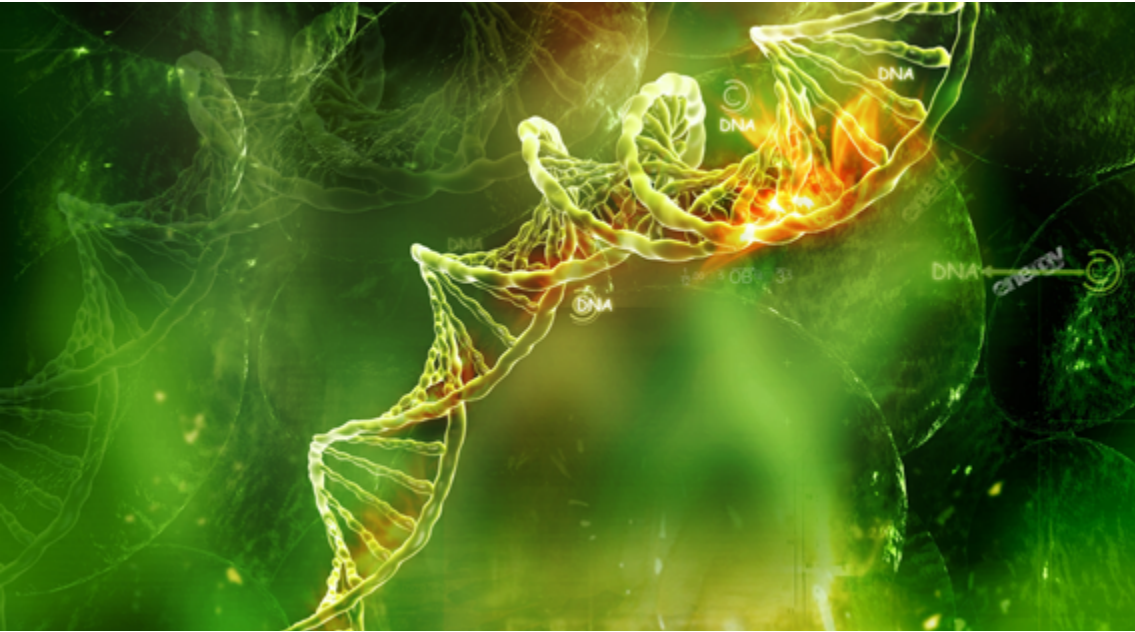

Pathological diagnosis
and follow up of patients with

Mature Lymphoid Neoplasms

References

*Swerdlow SH, Campo E, Harris NL, Jaffe ES, Pileri SA, Stein H, Thiele J
WHO Classification of Tumours of Haematopoietic and Lymphoid Tissues (Revised 4th edition)
IARC Lyon 2017



Biomnis

Eurofins Biomnis Ireland - Blackthorn Road
Unit 3, Sandyford Business Centre
Sandyford Business Park - D18 E528
Phone. 1800 303 349
sales@ctie.eurofinseu.com
www.eurofins-biomnis.ie

Eurofins Biomnis - International Division
17/19 avenue Tony Garnier
BP 7322 - 69357 LYON Cedex 07 - FRANCE